
Folding Walking Stick

Please read these instructions carefully and retain for future reference

Intended Uses

To help provide additional support when walking for longer periods of time. The Folding Walking Stick folds down with ease when not in use for easy storage and transportation.

Assembly and use

To open the Folding Walking Stic, firstly remove it from its storage pouch. Once removed all the folded shafts automatically unfold and snap into place. The stick can now be used to support the users weight when walking.

To fold the Folding Walking Stick pull each shaft away from the adjacent shaft and bend/fold and place back into the pouch.

Care & Cleaning

Before use, check to make sure that rubber tips are not ripped, worn or missing. Replace if necessary before use. Please inspect all parts carefully for cracks/breaks prior to use.

Clean with mild cleanse and water; dry thoroughly.

Size	GTIN
29-33"	5060639610044
33-37"	5060639610051

Specifications

Max user weight: 18 stone (114kg)

Handle height: 29-33" (73-84cm)
33-37" (84-94cm)

CAUTIONS

- When adjusting the height of the handle shaft, be sure the push button (A) fully protudes through the adjustment hole of the handle shaft to ensure a positive lock